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EN ATTENTION!!!

Do not use if packaging is opened or damaged.
Non sterile product. Reutilization is not recommended.
Single use recommended by the manufacturer.

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Nome comercial: Pilar Protético

EN Technical name: Prosthetic Abutment

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EN 1 – Identification

1.1 Technical name: Prosthetic Abutment

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Commercial Model: “UN II Morse Taper ST Abutment”

A) Description The UN II Morse Taper ST Abutment made by Implacil De Bortoli consists of a straight cylindrical abutment manufactured in titanium according to ASTM-F67. The abutments are for single use and have a collar height (a) of 0,8mm, 1,5mm, 2,5mm, 3,5mm, 4,5mm or 5,5mm. It has a Morse Taper fitting with angulation of 11 ° on its lower base that is used to adapt it to the type of implant and make it anti-rotation. The diameters (b) of 3.5mm and 4.5mm are compatible with all the available Morse Taper implant diameters. The thread (c) of 1.8 has a height of 1.5mm. It is used in the final production of the cemented prosthetic element, especially for single-tooth restorations.

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Raw Material:The raw material used for the UN II Morse Taper ST Abutment is Titanium according to ASTM F-67.

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EN 1.2 Technical name: Prosthetic Abutment

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Commercial model: “UN II Morse Taper ST Angled Abutment”

A) Description The UN II Morse Taper ST Angled Abutment of Implacil De Bortoli consists of an abutment with angulation of 15º or 25º and a fixation screw (a), manufactured in titanium according to ASTM-F136. The abutments are for single use and have a total height of 11.5mm and collars (b) of 0,8mm, 1,5mm, 2,5mm, and 3,5mm. It has a Morse Taper on its lower base that is used to adapt it to the implant and make it anti-rotational. Its diameters (c) are: 3.5mm and 4.5mm. The thread (d) of the fixation screw has a diameter of 1.8. It is used in the final production of the cemented prosthetic element, single-unit or multi-unit, where angulation correction is necessary.

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Raw Material: The raw material used for both the UN II Morse Taper ST Angled Abutment and the fixation screw is Titanium according to ASTM F136.

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EN 1.3 Technical name: Prosthetic Abutment

Trade Name: Universal II ST Prosthetic Abutment

Commercial model: “UN II Morse Taper ST Mini Tapered”

A) Product description

The UN II Morse Taper ST Mini Tapered Abutment of Implacil De Bortoli consists of a straight tapered abutment manufactured in titanium according to ASTM-F67, available in collar heights (a) of 0,8mm, 1,5mm, 2,5mm, 3,5mm, 4,5mm and 5,5mm. The abutments are for single use. It has a Morse Taper on its lower base that is used to adapt it to the morse taper implant and make it anti-rotational. Its diameter (b) is 4.8 mm. The thread (c) of 1.8 has a height of 1.5mm. It is used in the final production of the screwed prosthetic element, single-unit or multi-unit, especially in the aesthetic zone, in cases with limited interocclusal space.

B) Accompanying parts and accessories

The product is packed individually

Each pack consists of:

- one titanium prosthetic abutment

C) Technical specifications and features

Placement torque: 30N.cm

Dimensions

	Ø of the base (mm)	Collar height (mm)
UN II Morse Taper Mini Tapered	4,8.	0,8; 1,5; 2,5; 3,5; 4,5 e 5,5.

Raw Material: The raw material used for the UN II Morse Taper ST Mini Tapered Abutment is Titanium according to ASTM F-67.

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A) Description

The UN II Morse Taper ST Zero Collar Abutment made by Implacil De Bortoli consists of a straight cylindrical abutment and an adapter collar, both manufactured in titanium according to ASTM-F67. The abutments are for single use, has zero collar and taper height (a) of 4.0. Its diameter (b) of 3.5 or 4.5 mm is compatible with all the diameters of the commercially available Morse taper implants. The thread (c) of 1.8 has a height of 1.5mm. It has an adapter collar (d) for better stabilization of the prosthesis. It is used in the final production of the screwed or cemented prosthetic element, especially for single-tooth restorations where the implant is at the height of the cortical.

B) Accompanying parts and accessories

The product is packed individually

Each pack consists of:

- one titanium prosthetic abutment
- one adapter collar

C) Technical specifications and features

Placement torque: 30N.cm

	Ø of the base (mm)	Collar height (mm)	Height of Taper (mm)
UN II Morse Taper ST Zero Collar Abutment	3.5, 4,5	0.0	4.0

Raw Material: The raw material used for the UN II Morse Taper ST Zero Collar Abutment is Titanium according to ASTM F-67.

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Commercial Models: **“UN II HE ST Straight Aesthetic Taper”** and **“UN II HI ST Straight Aesthetic Taper”**

A) Description The UN II ST Straight Aesthetic Taper Abutment of Implacil De Bortoli consists of a straight tapered abutment manufactured in titanium according to ASTM-F67, and a fixation screw (a) of the abutment on the implant, manufactured in titanium according to ASTM-F136. The abutments are for single use, available with collar heights (a) of 1.0mm, 2.0mm and 3.0mm. Its upper base (c) has a hexagon of 3.30mm. It has an inner or outer hexagon on its lower base (d) that is used to adapt it to the type of implant and make it anti-rotational. Its diameters are related to the seating platforms of the commercially available implants: 3.5mm, 4.0mm and 5.0mm. The thread of the fixation screw is M1.8 for the platform implants of 3.5 mm and M2.0 for implants of 4.0 mm and 5.0 mm. It is used in the final production of the screwed prosthetic element, single-unit or multi-unit, especially in the aesthetic zone.

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Raw Material: Titanium is used as raw material for the UN II ST Straight Aesthetic Taper Abutment according to ASTM F-67 and Titanium is used for the Fixation Screw according to ASTM F136.

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EN 1.8 Technical name: Prosthetic Abutment

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Commercial Models: **“UN II HE ST Angled Aesthetic Taper”** and **“UN II HI ST Angled Aesthetic Taper”**

A) Description UN II ST Angled Aesthetic Taper Abutment of Implacil De Bortoli consists of a tapered abutment with angulation (a) of 17º or 30º, manufactured in titanium according to ASTM-F67, and a fixation screw (b) of the abutment on the implant, manufactured in titanium according to ASTM-F136. The abutments are for single use, available with collar heights (a) of 1.0mm, 2.0mm and 3.0mm. It has an inner or outer hexagon on its lower base (d) that is used to adapt it to the type of implant and make it anti-rotational. Its diameters are related to the seating platforms of the commercially available implants: 3.5mm, 4.0mm and 5.0mm. The thread (e) of the fixation screw is M1.8 for the platform implants of 3.5 mm and M2.0 for implants of 4.0 mm and 5.0 mm. It is used in the final production of the screwed prosthetic element, single-unit or multi-unit, especially in the aesthetic zone when angulation correction is necessary, enabling different horizontal rotation positions.

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Each pack consists of:

- one titanium prosthetic abutment
- one titanium fixation screw

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Raw Material: Titanium is used as raw material for the UN II ST Angled Aesthetic Taper Abutment according to ASTM F-67 and Titanium is used for the Fixation Screw according to ASTM F136.

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Commercial models: “UN II HE ST Anti-Rotation Straight Abutment” and “UN II HI ST Anti-Rotation Straight Abutment”

A) Description The UN II ST Anti-Rotation Straight Abutment of Implacil De Bortoli consists of a straight cylindrical abutment manufactured in titanium according to ASTM-F67, and a fixation screw (a) of the abutment on the implant, manufactured in pure titanium according to ASTM-F136. The abutments are for single use and have a total height (b) of 11.5 mm. It has an inner or outer hexagon on its lower base (c) that is used to adapt it to the type of implant and make it anti-rotational. Its diameters are related to the seating platforms of the commercially available implants: 3.5mm, 4.0mm and 5.0mm. The thread of the fixation screw is M1.8 for the platform implants of 3.5 mm and M2.0 for implants of 4.0 mm and 5.0 mm. It is used in the final production of the cemented prosthetic element, single-unit or multi-unit.

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Placement torque: 20N.cm

	Total Height (mm)	Ø of the base (mm)
UN II HE ST Anti-Rotation Straight Abutment	11.5	3.5, 4.0, 5.0
UN II HI ST Anti-Rotation Straight Abutment	11.5	3.5, 4.0, 5.0

Raw Material: Titanium is used as raw material for the UN II ST Anti-Rotation Straight Abutment according to ASTM F-67 and Titanium ASTM F136 is used for the Fixation Screw.

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	Angulação (°)	Ø da base (mm)
Munhão Angulado UN II HE ST	15,0, 25,0	3,5, 4,0, 5,0
Munhão Angulado UN II HI ST	15,0, 25,0	3,5, 4,0, 5,0

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Commercial models: “UN II HE ST Angled Abutment” and “UN II HI ST Angled Abutment”

A) Description UN II ST Angled Abutment of Implacil De Bortoli consists of a cylindrical abutment with angulation of 15° or 25°, manufactured in titanium according to ASTM-F67, and a fixation screw (b) of the abutment on the implant, manufactured in titanium according to ASTM-F136. The abutments are for single use and have a total height (a) of 11.5 mm. It has an inner or outer hexagon on its lower base (c) that is used to adapt it to the type of implant and make it anti-rotational. Its diameters are related to the seating platforms of the commercially available implants: 3.5mm, 4.0mm and 5.0mm. The thread (d) of the fixation screw is M1.8 for the platform implants of 3.5 mm and M2.0 for implants of 4.0 mm and 5.0 mm. It is used in the final production of the cemented prosthetic element, single-unit or multi-unit, where angulation correction is necessary.

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Raw Material: Titanium is used as raw material for the UN II ST Angled Abutment according to ASTM F-67 and Titanium ASTM F136 is used for the Fixation Screw.

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Commercial models: “UN II HE ST Ucla” and “UN II HI ST Ucla”

A) Description The UN II ST UCLAAbutment of Implacil De Bortoli consists of a straight cylindrical abutment manufactured in Titanium ASTM F67, and a fixation screw (a) of the abutment on the implant, manufactured in titanium according to ASTM-F136. The abutments are for single use and have a total height (b) of 11.5 mm. It is produced with an outer fitting on the base (c) that is used to adapt it to the implants of the Implacil System with inner fitting (HI) or outer fitting (HE). Its diameters are related to the seating platforms of the commercially available implants: 3.5mm, 4.0mm and 5.0mm. The thread (d) of the fixation screw is M1.8 for the platform implants of 3.5 mm and M2.0 for implants of 4.0 mm and 5.0 mm. It is commonly used in the production of the temporary cemented or screwed prosthetic element.

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Raw Material: Titanium is used as raw material for the UN II ST Ucla Abutment according to ASTM F-67 and Titanium ASTM F136 is used for the Fixation Screw.

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Commercial model "ST Aesthetic Taper Coping™" and "AR ST Aesthetic Taper Coping™"

A) Description
The ST Aesthetic Taper Coping and AR ST Aesthetic Taper Coping of Implacil De Bortoli consists of a straight cylindrical abutment manufactured in titanium according to ASTM-F67, and a fixation screw (a) of the abutment on the implant and a work screw (b), both manufactured in titanium according to ASTM-F136. The abutments are for single use and have a total height (c) of 14mm and diameter (d) of 4.8mm. It has an inner fitting (e) on its lower base that is used to adapt it to the Aesthetic Taper model of Prosthetic Abutments of the Implacil System. It also has a retention shoulder to support the prosthesis. The thread (f) of the fixation screw and work screw is M1.4.
The AR ST Aesthetic Taper Coping has a hexagon (g) to enable its locking. It is used in the production of the temporary screwed prosthetic element.
B) Accompanying parts and accessories
The product is packed individually
Each pack consists of:
- one titanium ST Aesthetic Taper Coping
- one titanium fixation screw
- one titanium work screw
C) Technical specifications and features
Placement torque: 20 N.cm
Dimensions

	Anti-Rotacional	Retenção
ST Aesthetic Taper Coping	no	yes
AR ST Aesthetic Taper Coping	yes	yes

Raw Material:
Titanium is used as raw material for the ST Aesthetic Taper Coping according to ASTM F-67 and Titanium ASTM F136 is used for the Fixation screw and Work screw.

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EN Physical principal and product technology fundamentals, applied to their operation and action indication:

The Prosthetic Abutments are mechanical components that work as a link between dental implant and prosthesis, offering support to the latter, and can correct height and angle between the implant and the tooth. They are fixed to the implant with a screw. The prosthesis elements can be screwed or cemented to the abutments, and it is up to the professional to decide the best method to be used in each situation.

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EN Optional, consumables and support materials

Optionals: Not applicable.
Consumables: Not applicable.
Manufacturing items for exclusive use of the company's products: All pieces described in these Usage Instructions are to be used exclusively on the Prosthetic Abutment.

Attention: The use of any piece, accessory or material not specified or described in these Usage Instructions is of the entire responsibility of the user.

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EN Special conditions for product transportation, storage and packaging:

Store at room temperature, in a dry place, away from light and heat. Avoid exposition to excessive heat and humidity. Do not subject the packaging to excessive weight to avoid compromising the sealing that ensures the product's sterility. Implacil De Bortoli Prosthetic Abutments must be transported in clean, dry places at room temperature. Transportation must be carried out properly to make sure the original packaging do not fall or get damaged and to avoid mechanical shocks that might damage the product.

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EN Directions for product usage

Implacil Prosthetic Abutments are packaged in aluminum sachets and a secondary packaging. Check to see if the packaging is in perfect conditions. The product must be rejected if packaging is damaged.

- Open the exterior packaging. Remove the sachet from the packaging and check if it is perfectly sealed.
- Remove the abutment from the sachet. The product is non-sterile. The sterilization process should be carried out by the professional according to the existing laws (NBR 14332) or any legislation or existing law that complements or supersedes it.
- Screw the abutment to the implant.
- Check the maximum torque to be applied in model Description (item C in these Usage Instructions).
- The abutments are previously sanitized in a classified room. If you feel the need the sanitize them again, follow the steps below:
 - Rinse the item with cold water.
 - Wash it with a neutral to slightly basic detergent.
 - Rinse with de-ionized or purified water by reverse osmosis.
- This procedure aims to completely eliminate microorganisms such as viruses, bacteria, microbes, fungi (vegetative or sporulating). After the drying process, the instruments should be sterilized by autoclaving, using temperature and exposition time according to the following method: Sterilization temperature: 132°C, Sterilization cycle: 20 minutes, and Drying cycle: 20 – 60 minutes. The surgical instruments should be placed inside the container (with holes on the lid and the bottom), or in a tray or box; as long as they are placed as to be prepared for the sterilization process. DO NOT open the autoclave to avoid rapid condensation; let the vapor out completely so that the Drying cycle is completed by itself. Clean the autoclave thoroughly and periodically, removing all dirt and rust residue formed (iron oxide).
- The product is non-sterile, and should be sterilized by the professional according to these steps: Sterilization temperature: 132°C, Sterilization cycle: 20 minutes and Drying cycle: 20 – 60 minutes. The items should be placed inside the container (with holes on the lid and the bottom), or in a tray or box; as long as they are placed as to be prepared for the sterilization process. DO NOT open the autoclave to avoid rapid condensation; let the vapor out completely so that the Drying cycle is completed by itself. Clean the autoclave thoroughly and periodically.

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EN Warnings and/or precautions to be adopted

Implacil Prosthetic Abutments can only be used by specialized professionals in the area. It is necessary to elaborate a prosthetic execution plan, observing the right diameter, angle and length for the adaptation of the prosthetic component to the implant in order to avoid future complications.
Implacil Prosthetic Abutments were designed for single use, and their reprocessing is prohibited. Do not use it in any way if the packaging is violated. The product is not sterile, so a sterilization procedure must be performed by the professional before any intervention on the patient.
The professional must observe the pressure exerted in the moment of application so as to not damage the item, and inform the patient about the necessity of periodic monitoring and avoiding physical and mechanical efforts in order not to submit the product to undue efforts, as well as the necessity of proper hygiene.

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EN Indication, purpose or use for the product

The abutments are indicated for the final preparation of single or multiple prosthetic elements, where functional and/or aesthetic rehabilitation is needed, and are fixed by a screw or dental procedure-specific cement. Any corrections needed between implant and tooth axis can be performed in either its vertical dimension or angle.

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EN Undesirable secondary or side effect and contraindications

The product application final result may be unsuccessful if an inadequate technique is used or if the patient is not subjected to the proper examinations and prosthetic planning. In these cases the product might be lost or fractured. Incorrect application of the product may cause side effects in the place where it was applied, such as pain, swelling, sensitivity, tissue reaction and infection.

Implacil Prosthetic Abutments are not indicated in cases of patients with chronic periodontal inflammation, inadequate oral hygiene or parafunctional habits, active intra-oral infection or in cases of immediate load where the primary stability of the implant is not adequate.

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EN Product safety and efficacy

Implacil Prosthetic Abutments are completely safe as long as the safety rules are observed in all procedures performed. The product was developed in non-toxic material for odontologic use. The product's characteristics will not change as long as it's stored and kept according to its usage indications.

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