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Use Instructions / Instrucciones de Uso / Istruzioni per l'Uso

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HEALING ABUTMENT FOR DENTAL IMPLANT AND SUPORT

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EN ATTENTION! Single Use Product. Do not use if the package is opened or damaged. Gamma radiation sterilized. Do not re-sterilize. For further safety, carefully read all directions herein before using this product.

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EN 1.1 Commercial Models “UN II Max HI Healing Abutment”

A) Description: The Implacil UN II HI Healing Abutment consists of a straight abutment, manufactured in commercially pure grade 2 titanium, in accordance with norm ASTM-F67. The Healing Abutments are designed for single use, supplied with collar heights (a) of 1.0mm; 2.0mm; 3.0mm; 4.0mm; 5.0mm; 6.0mm and 7.0mm. Its upper base (b) has a 1.17 hexagon for connection with the insertion driver. Its lower base has an M2.0 thread (c) to be screwed on to the corresponding implant. It offers heights and formats that are compatible with the different abutments available and substitutes them during the cicatrization period, to allow the gengival tissue to obtain the shape required for the desired esthetic profile. The product's diameters relate to the seating platforms for the available implants: 3.5mm; 4.0mm and 5.0mm.

B) Dimensions: UN II HI Healing Abutment: Ø from base (mm) 3,5; 4,0; 5,0. Ø maximum (mm): 4,0; 4,5; 5,5. Collar Height (mm) 1,0; 2,0; 3,0; 4,0; 5,0; 6,0; 7,0.

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EN 1.2 Commercial Models “UN II Max HE Healing Abutment”

A) Description: The UN II HE Healing Abutment made by Implacil consists of a straight abutment, manufactured in commercially pure, grade 2 titanium, in accordance with norm ASTM-F67. The Healing Abutments are designed for single use, supplied with collar heights (a) of 1.0mm; 2.0mm; 3.0mm; 4.0mm; 5.0mm; 6.0mm and 7.0mm. Its upper base (b) has a 1.17 hexagon for connection with the insertion driver. Its lower base has an M2.0 thread (c) to be screwed on to the corresponding implant. It offers heights and formats that are compatible with the different abutments available and substitutes them during the cicatrization period. It allows the gengival tissue to obtain the shape required for the desired esthetic profile. The product's diameters relate to the seating platforms for the available implants: 3.5mm; 4.0mm and 5.0mm.

B) Dimensions: UN II HE Healing Abutment: Ø from base (mm) 3,5; 4,0; 5,0. Ø maximum (mm): 4,0; 4,5; 5,5. Collar Height (mm) 1,0; 2,0; 3,0; 4,0; 5,0; 6,0; 7,0.

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EN 1.3 Commercial Models “UN II Max Morse Taper Healing Abutment”

A) Description: The UN II Morse Taper Healing Abutment made by Implacil consists of an abutment made in commercially pure, grade 4 titanium, in accordance with norm ASTM –F67. The Healing Abutments are designed for single use, supplied with collar heights (a) of 0.80mm; 1.50mm; 3.0mm; 4.5mm and 6.0mm. Its upper base (b) has a 1.17 hexagon for connection with the insertion driver. Its lower base has an M1.8 thread (c) to be screwed on to the corresponding implant. It offers heights and formats that are compatible with the different abutments available and substitutes them during the cicatrization period. It allows the gengival tissue to obtain the shape required for the desired esthetic profile. Its diameters are compatible with all of the diameters for the commercially available Morse Taper implants.

B) Dimensions: UN II Morse Taper Healing Abutment: Ø maximum (mm): 3,3; 4,5. Collar Height (mm) 0,8; 1,5; 2,5; 3,5; 4,5; 5,5.

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EN 1.4 Commercial Models “UN II Max HI Emergency Profile Healing Abutment”

A) Description: The UN II HI Emergency Profile Healing Abutment made by Implacil consists of a straight abutment, manufactured in commercially pure, grade 2 titanium, in accordance with norm ASTM-F67. The Healing Abutments are designed for single use, supplied with collar heights (a) of 1.0mm; 2.0mm; 3.0mm; 4.0mm; 5.0mm; 6.0mm and 7.0mm. Its upper base (b) has a 1.17 hexagon for connection with the insertion driver. Its lower base has an M2.0 thread (c) to be screwed on to the corresponding implant. It offers heights and formats that are compatible with the different abutments available and substitutes them during the cicatrization period, to allow the gengival tissue to obtain the shape required for the desired esthetic profile. The product's diameters relate to the seating platforms for the available implants: 3.5mm; 4.0mm and 5.0mm.

B) Dimensions: UN II HI Emergency Profile Healing Abutment: Ø from base (mm) 3,5; 4,0; 5,0. Ø maximum (mm): 5,5; 6,0; 7,0. Collar Height (mm) 1,0; 2,0; 3,0; 4,0; 5,0; 6,0; 7,0.

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EN 1.5 Commercial Models “UN II Max HE Emergency Profile Healing Abutment”

A) Description: The UN II HE Emergency Profile Healing Abutment made by Implacil consists of a straight abutment, made in commercially pure, grade 2 titanium, in accordance with norm ASTM-F67. The Healing Abutments are designed for single use, supplied with collar heights (a) of 1.0mm; 2.0mm; 3.0mm; 4.0mm; 5.0mm; 6.0mm and 7.0mm. Its upper base (b) has a 1.17 hexagon for connection with the insertion driver. Its lower base has an M2.0 thread (c) to be screwed on to the corresponding implant. It offers heights and formats that are compatible with the different abutments available and substitutes them during the cicatrization period, to allow the gengival tissue to obtain the shape required for the desired esthetic profile. The product's diameters relate to the seating platforms for the available implants: 3.5mm; 4.0mm and 5.0mm.

B) Dimensions: UN II HE Emergency Profile Healing Abutment: Ø from base (mm) 3,5; 4,0; 5,0. Ø maximum (mm): 5,5; 6,0; 7,0. Collar Height (mm) 1,0; 2,0; 3,0; 4,0; 5,0; 6,0; 7,0.

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EN 1.6 Commercial Model “UN II Max Morse Taper Emergency Profile Healing Abutment”

A) Product description

The UN II Max Morse Taper Emergency Profile Healing Abutment made by Implacil De Bortoli consists of a straight abutment manufactured in titanium according to ASTM-F67. The healing abutments are designed for single use, supplied with collar heights (a) of 0.8mm; 1.50mm; 2.50mm; 3.0mm; 4.50mm and 6.50mm. Its upper base (b) has a 1.17 hexagon for connection with the insertion driver. Its lower base has an M2.0 thread to be screwed to the platform implants 4.0 mm and 5.0 mm, and the lower base thread for 3.5 platform implants is M1.8 (c). It has heights and shapes compatible with the different abutments available, and substitutes them during the healing period to allow the gengival tissue to obtain the shape required for the desired esthetic profile. Its diameters are related to the seating platforms of the commercially available implants: 3.5mm; 4.0mm and 5.0mm.

UN II Max Morse Taper Emergency Healing Abutment

B) Accompanying parts and accessories

The product is packaged individually

Each pack consists of:

- one titanium Healing Abutment

C) Technical specifications and features

Placement Torque: 10 Ncm (See Test Report no. RE 001-2015)

Dimensions

	Ø of the base (mm)	Collar height (mm)
UN II Max Morse Taper Emergency Healing Abutment	5,50; 6,30	0,80; 1,50; 2,50; 3,50; 4,50; 5,50

Raw Material:

The raw material for the "UN II Max Morse Taper Emergency Healing Abutment" is commercially pure ASTM F-67 Titanium.

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EN 1.8 Commercial Model “UN II Max HE ST Healing Abutment”

A) Product description

The UN II Max HE ST Healing Abutment made by Implacil De Bortoli consists of a straight abutment manufactured in titanium according to ASTM-F67. The healing abutments are designed for single use, supplied with collar heights (a) of 1,0mm; 2,0mm; 3,0mm; 4,0mm; 5,0mm; 6,0mm and 7,0mm. Its upper base (b) has a 1.17 hexagon for connection with the insertion driver. Its lower base has an M2.0 thread to be screwed to the platform implants 4,0 mm and 5,0 mm, the lower base thread for 3.5 platform implants is M1.8 (c), and when compared to the UN II Max HE Healing Abutment the thread length is smaller by 1mm. It has heights and shapes compatible with the different pillars available, and it replaces them during the healing period. It allows the gingival tissue to obtain the appropriate shape for the desired esthetic profile. Its diameters are related to the seating platforms of the commercially available implants: 3.5mm; 4,0mm and 5,0mm.

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Placement Torque: 10 Ncm (See Test Report no. RE 003-2015)

UN II Max HE ST Healing Abutment	Ø of the base (mm)	Collar height (mm)
	3,5; 4,0; 5,0	1,0; 2,0; 3,0; 4,0; 5,0; 6,0; 7,0

Raw Material:

The raw material for the “UN II Max HE ST Healing Abutment” is commercially pure ASTM F-67 Titanium.

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EN 1.9 Commercial Model “UN II Max Morse Taper ST Healing Abutment”

A) Product description

The UN II Max Morse Taper ST Healing Abutment made by Implacil De Bortoli consists of a straight abutment manufactured in titanium according to ASTM-F67. The healing abutments are designed for single use, supplied with collar heights (a) of 0,80mm; 1,50mm; 2,50mm; 3,50mm; 4,50mm and 5,50mm. Its upper base (b) has a 1.17 hexagon for connection with the insertion driver. Its lower base has an M1.8 (c) thread to be screwed to the corresponding Implant, and when compared to the UN II Max Morse Taper Healing Abutment the thread length is smaller by 1mm It has heights and shapes compatible with the different pillars available, and it replaces them during the healing period. It allows the gingival tissue to obtain the appropriate shape for the desired esthetic profile. Its diameters are compatible with all diameters of the commercially available Morse Taper implants.

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Raw Material:

The raw material for the “UN II Max Morse Taper ST Healing Abutment” is commercially pure ASTM F-67 Titanium.

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EN 1.10 Accompanying parts and accessories : The product is packaged individually. Each package contains a titanium Healing Abutment.

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EN 2. Physical principal and technological fundaments for the product, applied for its functionality and action: Implacil Healing Abutments are mechanical components used to model the gingival tissue during the period of waiting before the implant is fitted. They are fixed to the implant using a screw. They are also indicated for protecting Implant Implants, preventing the soft tissue from closing over the implant and promoting a perfect adaptation of the previously selected prosthetic abutments.

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EN 3. Optional extras, consumer materials and support materials: Optional extras: Non-applicable. Consumer Materials: Non-applicable. Support material: Instructions for use. Manufactured items for exclusive use with the Company's products: All of the components described in this Instruction for Use are for exclusive use with the Healing Abutment product. Attention: The utilization of any component, accessory or material not specified or provided for in these instructions for use is the sole responsibility of the user.

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EN 4. Special conditions for product transport, storage and packing: Storage: Store away from direct sunshine and rain and in its original packaging. Transport: During transport, protect the packaging from direct rain. Packing: Keep away from direct sunshine and rain.

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• Parafuse-o no implante.

EN 5. Instructions for use of the product:

• Implacil Healing Abutments come in a blister pack sealed with Tyvek. The blister is placed inside a cardboard carton.

• Check that the packaging is in perfect condition. Do not use the product if the packaging is damaged.

• Open the outer packaging. Remove the blister from the box and check that it is perfectly sealed.

• Remove the Healing Abutment from the blister pack.

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EN 6. Warnings and/or precautions to be taken: The utilization of Implacil Healing Abutments must be carried out by specialized dental surgeons. A conscientious routine of oral hygiene at home and regular professional treatment are required in order to conserve the restorations in good condition. Implacil Healing Abutments are designed for single use and their re-utilization is strictly prohibited. Under no circumstances must they be used if the packing seal is broken. The dental surgeon must pay attention to the amount of force used when applying the product so as not to damage it and must also inform the patient of the need for periodic monitoring that avoids physical and mechanical effort in order to avoid subjecting the product to undue strain.

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EN 7. Indication, purpose or use to which the product will be put: Implacil Healing Abutments are indicated for the cicatrization of the soft tissue (gums) protecting Implacil Implants, preventing the soft tissue from closing over the implant and promoting a perfect adaptation of the previously selected prosthetic abutments, while waiting for the prosthetic component to be made.

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EN 8. Undesired secondary or side effects and contraindications: The end result of the application of the product may be unsuccessful if inappropriate techniques used and this may lead to loss or fracturing of the product. Incorrect application of the product may lead to side effects in the region where it is applied, such as: pain, swelling, sensitivity, tissue reaction and infection. The utilization of Implacil Healing Abutments is contraindicated in cases where the patient presents chronic periodontal inflammation and inadequate oral hygiene and parafunctional habits, active intra-oral infection and in cases of Immediate Loading where the primary stability of the implant is insufficient.

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